# Roger™ Touchscreen Mic

Quick setup guide

Schnellstartanleitung

Guide d'installation rapide

röger

PHONAK

Components  
Komponenten  
Composants

Power supply / charger  
Ladegerät  
Alimentation

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
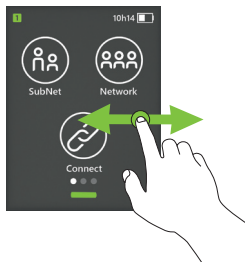
## Connecting Verbinden Connecter

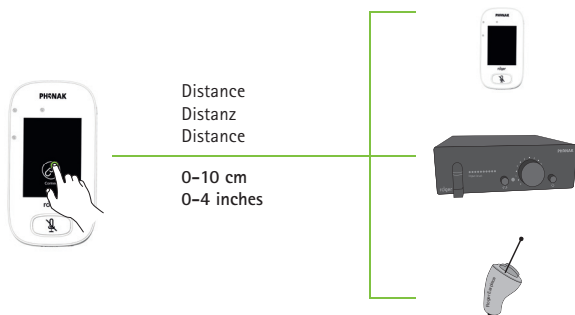To connect a Roger Touchscreen Mic to a Roger BaseStation, press either:

- **Connect** on the Roger BaseStation or
- **JoinNet** on the Roger Touchscreen Mic

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Reset  
Zurücksetzen  
Réinitialiser

## Roger Earpiece settings - ENG

Use this function to configure the following Roger Earpiece data:

- **Name** (for easy naming and identification of individual earpieces)
- **EasyGain** (adjust volume level individually)
- Ensure only one Roger Earpiece is located within 10 cm / 4 inches of Roger Touchscreen Mic.
- Select **Settings > Roger receiver settings**
- Select **Name** to change the Roger Earpiece's name
- Select **EasyGain level** to change the Roger Earpiece's volume

The **EasyGain** value will only act on a Roger Earpiece which is within the 10 cm / 4 inch Roger Touchscreen Mic range. Depending on the volume setting of the Roger BaseStation, the **EasyGain** setting may have no effect. Please see the Roger Touchscreen Mic full user-guide for further information.

To change another Roger Earpiece's **Name** or **EasyGain** settings, return to the settings menu and select **Settings > Roger receiver settings** when the new Roger Earpiece is within 10 cm / 4 inches of the Roger Touchscreen Mic.

### Read Roger Earpiece info

- Ensure only one Roger Earpiece is located within 10 cm / 4 inches of Roger Touchscreen Mic
- Select **Check** icon
- Following parameters can be read:
- Name / Device type / Firmware version

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Phonak Communications AG